




Second Skin

CUSTOMISED COMPRESSION GARMENTS & SPLINTS

Washing tips

SOILED GARMENTS & SPLINTS (TOILETING)*

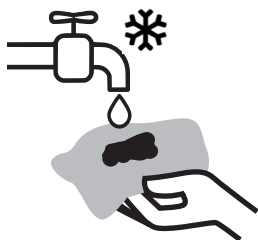

- As soon as possible remove any excess matter - **trying not to rub any into the garment or splint.**
- **Run the soiled area under COLD running water** to help lift any substance from the soiled area. **DO NOT** use warm or hot water as this will lock in the discolouration and smell. You can soak the affected area in cold water also.



- Once you have lifted most of the substance from the stained area, you can now wash the garment or splint, in the washing machine on a warm cycle.

Use **washing powder** and **add ½ cup of vinegar to the wash** (this will help kill the germs). You can add a couple of drops of essential oils to the vinegar to give a pleasant smell – citrus works well.



- Once the garment or splint is clean, **dry flat in an open ventilated area**

** Please take the necessary precautions, such as wearing gloves, when coming into contact with bodily fluids*